
DK	Guldmann Positionslås – Træningspakke	3
GB/US	Training kit for Guldmann Positioning lock	6
DE	Trainings Kit für Feststellbremse	9
SE	Tränings kit för Positioneringslås	12
NO	Treningspakke for posisjonslås	15
FR	Accessoires d'entrainement pour positionning lock	18
IT	Pacchetto training per Positioning Lock	21

Manual – Version 102.00

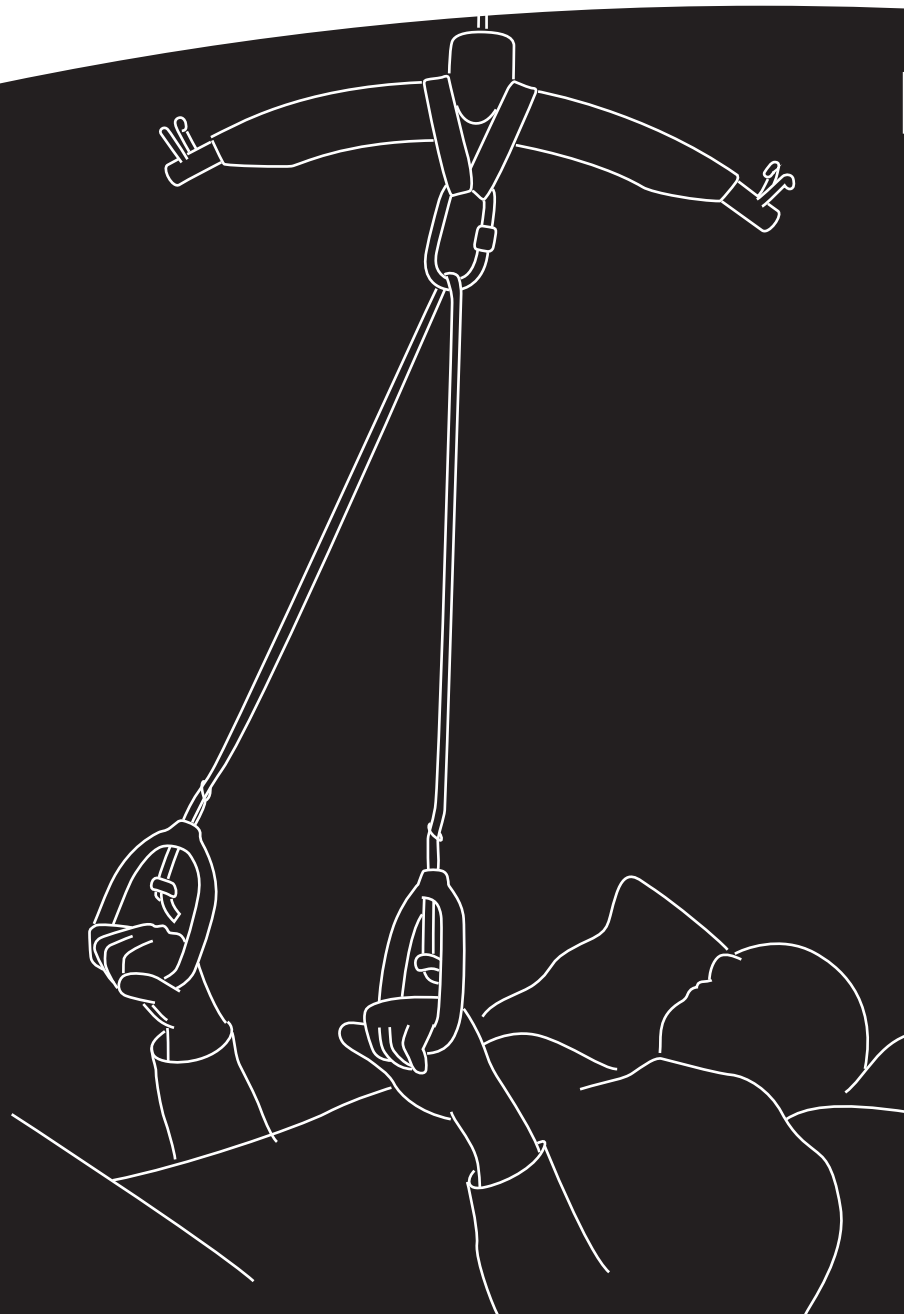MD CE

Guldmann™

Vers. 102.00

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Vers. 102.00

293560 Training kit for Guldmann Positioning lock

*The kit contains:*

- 558136 A. 30 m training tube: Orange (light)  
 558137 B. 30 m training tube: Lime green (medium)  
 558138 C. 30 m training tube: Plum (hard)  
 558139 D. 2 hand grips for training tube  
 561628 E. Snap hook  
 29356 F. 2 connecting straps  
 561629 G. 2 elastic lines: Blue (light) and black (hard)  
 28467 H. Multi support sling

*A-H can be ordered separately*

## 1.00 Using the training accessories Purpose and use

### 1.01 Manufacturer

V. Guldmann A/S  
 Graham Bells Vej 21-23A  
 DK-8200 Aarhus N  
 Tel. + 45 8741 3100  
 www.guldmann.com

### 1.02 Intended purpose

The training kit for the GH3 ceiling hoist system and Position Lock combination is intended for physiotherapy and rehabilitation of muscles and body segments to aid patients in early resistive training to minimize risks typically associated with long-term immobility.

### 1.03 Area of use

The accessories can be used at hospitals, assisted living facilities, physical therapy centres and in the patient's home for the physical therapy of people with reduced functionality.

### 1.04 Conditions for use

When the accessories are used, it is important that:

- the accessories are used only by qualified personnel or individuals who have received instruction in how to use the training kit.
- the accessories are suitable for training a person who is capable of actively participating in the process, and the user must have good communication skills.
- the accessories can be used on a person in both a lying down and seated position.
- the carer is attentive to the user's well-being during the training.
- the accessories must not be used for lifting and handling of people.
- the accessories are used with a GH3 lifting module combined with the Positioning lock.

### 1.05 Warnings

- Do not exceed max load as described in 1.06.
- Before use, the training equipment must be examined for omissions, wear and defects such as cracks, holds, tears or rips. If it does not meet the inspection requirements, it must be discarded if necessary.
- Do not modify this equipment without authorization of the manufacture.
- Any serious incident that occurred in relation to this device should be reported to the manufacturer and the local competent authority.

### 1.06 Using the training accessories

The training kit can be used in multiple ways.

1. Attaching the straps to the hanger, connecting the straps with the safety hook, gives you the opportunity for excising with both tubes and multi-support sling.

2. Depending on the height of the ceiling and to increase the width of the multi-support sling, the elastic line can also be attached from one hanger hook to the other.

3. For heavier patients, it is recommended to use two elastic lines, one from each hook of the hanger, giving you extra strength and increasing the max load.

For maximum load capacity, use two black elastic lines, one from each hook of the hanger, and benefit from a max load of 266 kg. 133 kg when only one elastic line is used. For the lighter elastic line, the blue version, you will gain a max load of 133 kg using the double elastic line technique, and 66 kg by using the single line only.

#### Max load for elastic lines

Blue	66 / 133 kg
Black	133 / 266 kg

### 2.00 Cleaning

Clean the training equipment with a damp cloth using ordinary household detergents. Chemicals or an autoclave must not be used when cleaning the training equipment. See specific cleaning instructions for the Multi support sling in separate manual.

### 3.00 Lifetime

The life of the training equipment is individual and depends on how it is used.

### 4.00 EU Declaration of conformity

The products is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device Class I.

### 5.00 Environmental policy statement – V. Guldmann A/S

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

---

## 6.00 Warranty and service conditions

---

### A. Warranty

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

### Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and from any and all claims or liability arising as a result of equipment malfunction or misuse.

---

### B. Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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GB/US  
Training kit for  
Guldmann Positioning lock

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## 1A. GB/US

*When using the training accessories, it is possible to choose between performing the training with the lifting hanger attached (A) or with the lifting hanger not attached (B).*

### A: Mounting of training equipment with lifting hanger attached

Place the two connecting straps on the hanger and attach the snap hook in both straps as shown in the picture.

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**Guldmann Sverige AB**
**Guldmann Srl**
**Guldmann Inc.**